


Hook Saver Strap Installation

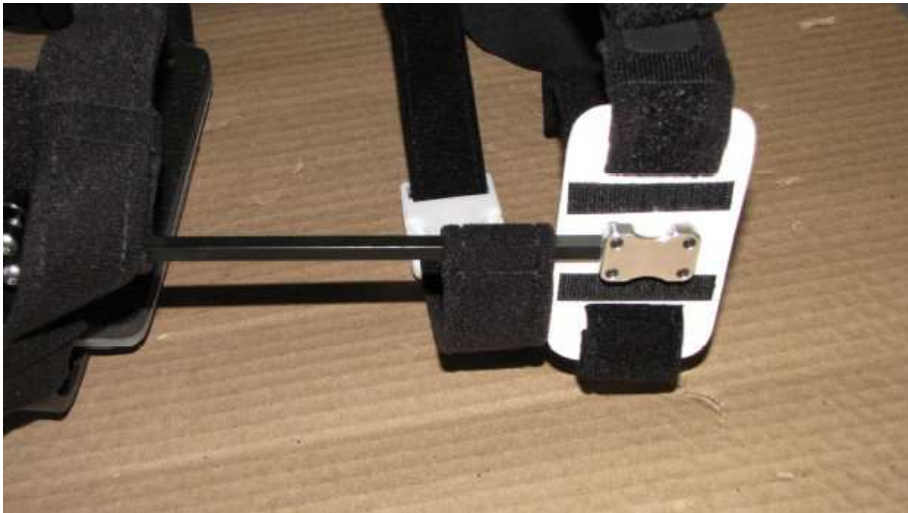

Remove the neoprene cover. Then loosen but do not remove the allen screws inside of the thoracic plastic. Slide the plastic off as shown above.



Replace and retighten the thoracic plastic. Slide the new strap proximally with buckle to the posterior of the brace.



Replace the neoprene pad with both hook tabs as shown. One of these holds the hook saver strap.

Hook Saver Strap Installation



Place a new Velcro tab onto the strap. This will hold the neoprene down as you fold it in place.



The white buckle goes posterior of the child and around the mast. Adjust it to be fairly tight around the mast. This stops forward leaning patients from bending the hook. Encourage parents to use it every time.