

TAOS Wall Bumper Installation

Bumper kit contains 2 wheel units. They are not left/right specific. You need a 3/16 allen wrench to install. This is the same wrench you use for the ankle bolt on the orthotic.



Step 1: Loosen the bolts and separate as shown.



Step 2: place wheel/clamp with bolt past tube.



Step 3: Bring lower part of clamp to position.



Step 4: Hand tighten.



Step 5: Level the wheel. Level equals less wall marking.
(You can use a level as shown or just use your eye to do it.)



Step 6: Tighten the bolts from below.